




I.V. HOUSE®
Protection Over and Above®



I.V. House UltraDressing®
Intravenous Site
Protectors




LATEX FREE

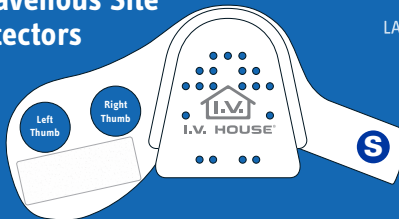

Item 730S (illustrated above)

Available in Small, Medium and Large Sizes
Designed To Fit Either Hand

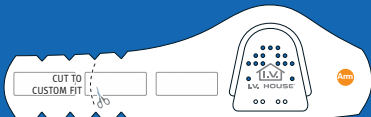


Item 730Arm

Pediatric and Adult Patients:

- Wrist, Forearm, Antecubital Space or Upper Arm

Neonates and Infants:

- Hand, Wrist, Forearm, Antecubital Space or Upper Arm
- Foot and Saphenous Placement

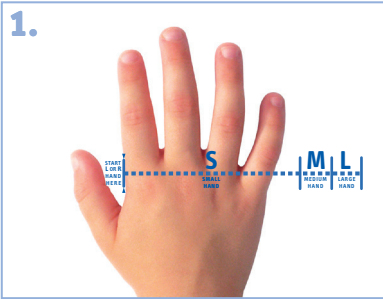

Pediatric Hand Directions

Each dispenser box includes twenty-four hand measurement charts. Simply place the patient's hand on the left margin of the chart to determine if they need a small, medium, or large size I.V. House UltraDressing.

The measurement is taken from the proximal interphalangeal joint of the 1st and 4th finger (knuckle to knuckle) and will assign the correct size needed.

Insert the IV catheter and securely attach an IV extension set or IV tubing. Secure IV site with transparent dressing, tape, or adhesive anchor. Loop the IV tubing away from the catheter hub and toward the IV fluid source.

Make sure the loop of tubing or extension set is small enough to fit entirely under the inner curve of the domed end of the I.V. House UltraDressing.

(Step-by-step directions for IV site taping are also available at www.ivhouse.com)

Position the I.V. House UltraDressing over the looped tubing and catheter hub.

Wrap the I.V. House UltraDressing around hand.

Close with Velcro®.

Inspect the IV site according to nursing practice guidelines: release the closure tab under the label and lift the dome to visualize the entire IV site. After viewing the entire IV site, re-close the device.

www.ivhouse.com
800.530.0400



800-530-0400

IVinfo@ivhouse.com

Learn More: visit our website: www.ivhouse.com

Instructive Videos / Usages / Directions / Product Details and Ordering Information