



I.V. House UltraDome® IV Site Protector 750LFP or 727SFP / Directions

Single Use Only. The I.V. House UltraDome can be used to protect any peripheral IV site (hand, foot, AC, forearm, scalp).


1. Tape the loop of tubing or extension set small enough to fit entirely under the inner curve of the domed end of the I.V. House UltraDome.

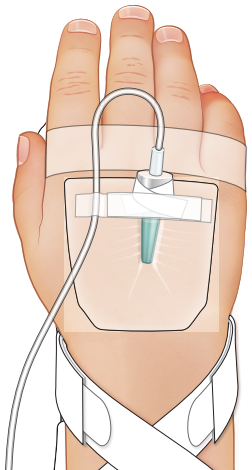


2. Place I.V. House UltraDome over the catheter hub and transparent dressing.

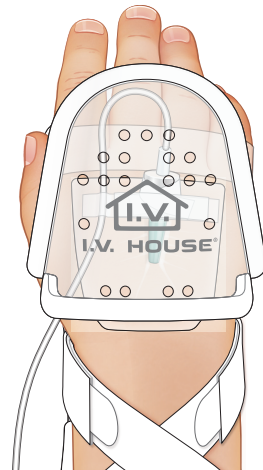


3. Tear off an 8" strip of tape and fold one end over, creating a tab for easy removal.

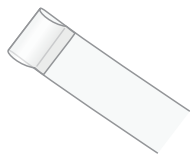

Tape across the I.V. House UltraDome and onto armboard to hold the protective dome in place. Allow the tubing to exit the open end.

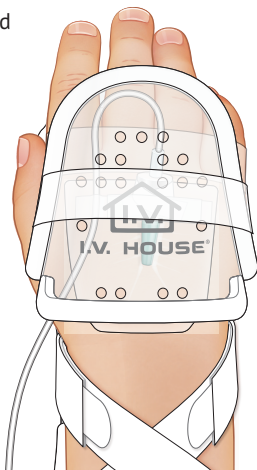


4. **Alternate:** For extra securement, place 2 pieces of tape over the top of the dome in an X configuration, affixing the dome to the armboard.

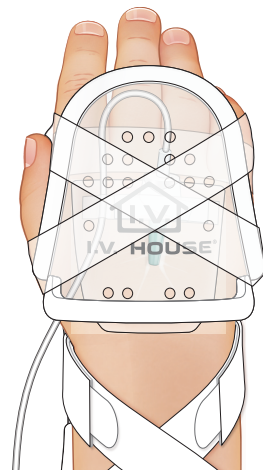

5. Using the tab created earlier, lift the tape from one side to assess the IV site.

Touch, Look, and Compare with the opposite extremity.

Change the dome each time a new IV is inserted.

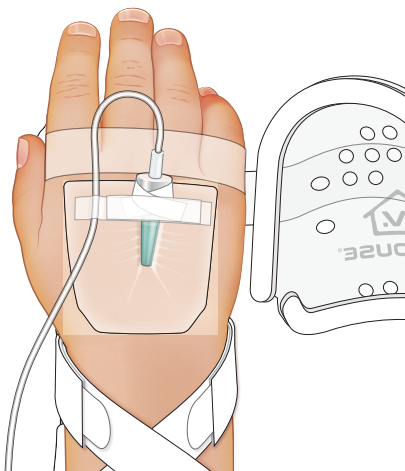

6. **Puppy Sticker:** Remove the crack and peel backing from the puppy sticker and apply the sticker to the dome.

Tell the patient that the puppy likes to sleep on the "house" and to leave her alone.

