



**I.V. HOUSE®**

Protection Over and Above®

**727SFP**

# 727SFP I.V. House UltraDome® IV Site Protector

**1 Small Foam Padded Dome IV Site Protector • 1 Sheet of 4 Puppy Stickers**

Directions for use on back of insert

Manufactured for I.V. House, Inc., 2072 Concourse Dr., St. Louis, MO 63146 USA

IVinfo@ivhouse.com (E-mail)

www.ivhouse.com

800-530-0400 (Toll Free)

314-453-9200 (Phone)

314-453-9576 (Fax)

All Rights Reserved

Made in USA



(01)00686004000019



Wellkang Ltd, Enterprise Hub,  
NW Business Complex, 1 Beraghmore Rd,  
Derry, BT48 8SE, Northern Ireland



CATALOG ITEM: **REF** 727SFP

**LOT** 2024-04-01-727SFP REV 0424



SINGLE USE



MRI SAFE



LATEX FREE



DEHP  
DEHP FREE



BPA FREE

# 727SFP I.V. House UltraDome® Directions

## Single Use Only

1. Following IV insertion, **tape the loop of tubing** small enough to fit entirely inside the inner curve of the dome.
2. **Place an I.V. House UltraDome** over the catheter hub and transparent dressing.
3. **Tape across the I.V. House UltraDome** to hold the protective dome in place. Allow the tubing to exit the open end.
4. Lift the tape from one side to **inspect the IV site** according to your nursing practice guidelines.
5. **Change the I.V. House UltraDome** each time a new IV is inserted.

## Puppy Sticker

Remove the crack and peel backing from the puppy sticker and apply the sticker to the dome.

Tell the patient that the puppy likes to sleep on the “house” and to leave her alone.



For Detailed Directions  
[www.ivhouse.com/how](http://www.ivhouse.com/how)