

**WRIST / INFANT / 2.5-12 KG**

**BASIC SPLINT**

**APPLY WITH TAPE**

## TLC<sup>®</sup> Wrist Splint

Touch/Look/Compare

Basic Splint—Apply with Tape



## 939S-Basic

1 Small (Infant) Size  
Foam Padded Wrist Splint

Fits most infants ages 0-12 months,  
or weighing 2.5-12 kg

CATALOG ITEM:

**REF** 939S-Basic



(01)00686004000101

**LOT** 2024-04-01-939S-Basic REV 0424



**I.V. HOUSE<sup>®</sup>**

**Protection Over and Above<sup>®</sup>**

# TLC<sup>®</sup> Wrist Splint—Apply with Tape

## Directions: Single Use Only—Fits either hand

- 1. Apply 3 strips of anchor tape to the foam padding** on the back of the splint, one across the width of the splint at each end, and one across the center wrist support.
- 2. The “fingers/wrist” label indicates proper placement.** Remove the label before applying the splint.
- 3. Place the splint under the patient’s forearm,** with the heel of the hand landing in the slight bend of the device. The fingers should curl naturally over the end of the splint, leaving the hand in a functional position.
- 4. Secure the splint with tape.** Place a ½” strip of tape across fingers and secure both ends to the anchor strip on the bottom of the splint. Fold under one end of tape to create a tab for easy removal.
- 5. Repeat tape application at the wrist and forearm,** using double-backed tape to prevent epidermal stripping.
- 6. Notes:** Do not cover openings on back side of the splint with tape. Covering the openings will obstruct the view of the palm and forearm.

For Detailed Directions  
[www.ivhouse.com/how](http://www.ivhouse.com/how)



Manufactured for I.V. House, Inc.  
2072 Concourse Dr.  
St. Louis, MO 63146 USA  
800-530-0400 (Toll Free)  
314-453-9200 (Phone)  
314-453-9576 (Fax)

Patents Pending  
All Rights Reserved  
Made in USA  
[www.ivhouse.com](http://www.ivhouse.com)  
[IVinfo@ivhouse.com](mailto:IVinfo@ivhouse.com)



SINGLE USE



MRI SAFE



LATEX FREE



DEHP  
DEHP FREE



BPA FREE



Wellkang Ltd, Enterprise Hub,  
NW Business Complex, 1 Beraghmore Rd,  
Derry, BT48 8SE, Northern Ireland

