

ELBOW / INFANT / 6-12 KG

TLC[®] Elbow Splint with Straps

Touch/Look/Compare



959S-Ultra

1 Small (Infant) Size Foam Padded Elbow Splint
with Three Straps and Velcro[®] Closure

Fits most infants ages 4-12 months,
or weighing 6-12 kg
Small splint, longer straps

CATALOG ITEM:

REF 959S-Ultra



(01)00686004000347

LOT 2024-04-01-959S-Ultra REV 0424



I.V. HOUSE[®]

Protection Over and Above[®]

TLC[®] Elbow Splint with Straps

Directions: Single Use Only—Fits either arm

1. **Unroll the TLC Elbow Splint straps** and identify the smooth concave surface of the splint. This concave surface will be placed against the patient's arm.
2. The **"forearm/upper arm" label indicates proper placement**. Remove the label before applying the splint.
3. **Place the splint under the patient's arm**, with the narrow end of the splint toward the hand and the wide end above the elbow. The point of the elbow should align with the center opening. **We recommend a thumbs-up application** whenever possible.
4. **Wrap the first strap around the forearm**. Thread the Velcro[®] tab into slot and pull it through. Fold the strap around the outer edge of the splint and back onto itself. Secure with Velcro tab.
5. **Repeat with the other two straps**, securing each with the Velcro tab.
6. **Note:** Slip one finger beneath each strap to ensure the fabric is not so tight that it impedes circulation or infusion of IV fluids.
7. **If the elbow strap obscures the catheter insertion site**, consider a crisscross strap configuration instead. Put the elbow strap through the upper arm slot and the upper arm strap through the elbow slot.

For Detailed Directions
www.ivhouse.com/how



Manufactured for I.V. House, Inc.
2072 Concourse Dr.
St. Louis, MO 63146 USA
800-530-0400 (Toll Free)
314-453-9200 (Phone)
314-453-9576 (Fax)

Patents Pending
All Rights Reserved
Made in USA
www.ivhouse.com
IVinfo@ivhouse.com



SINGLE USE



MRI SAFE



LATEX FREE



DEHP
DEHP FREE



BPA FREE



Wellkang Ltd, Enterprise Hub,
NW Business Complex, 1 Beraghmore Rd,
Derry, BT48 8SE, Northern Ireland

