


The 959XL-Ultra Elbow Splint will fit most patients who require a **small adult** or **standard adult** size blood pressure cuff.

If you would like more detailed information to choose the correct size, refer to steps below.

1. Measure circumference of patient's upper arm.

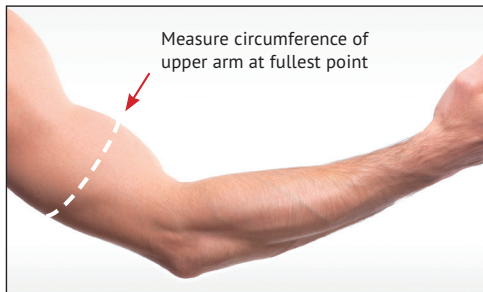

2. 959XL-Ultra TLC Elbow Splint will fit patients within parameters listed in chart below:

TLC Elbow Splint Size	Fits Arm Circumference	
	cm	inch
<b>XL</b> 959XL	24 – 34	9.4 – 13.4



Video Directions: [www.ivhouse.com/how](http://www.ivhouse.com/how)

Manufactured for I.V. House, Inc.  
 2072 Concourse Dr.  
 St. Louis, MO 63146 USA  
[IVinfo@ivhouse.com](mailto:IVinfo@ivhouse.com) (E-mail)  
[www.ivhouse.com](http://www.ivhouse.com)  
 800-530-0400 (Toll Free)  
 314-453-9200 (Phone)  
 314-453-9576 (Fax)

Patents Pending  
 All Rights Reserved  
 Made in USA

**EC REP** Wellkang Ltd, Enterprise Hub,  
 NW Business Complex, 1 Beraghmore Rd,  
 Derry, BT48 8SE, Northern Ireland



## TLC® Elbow Splint with Straps

Touch/Look/Compare



### 959XL-Ultra

1 Extra-Large (Youth/Adult) Size Foam Padded Elbow Splint with Two Straps and Velcro® Closure

Fits arms 24–34 cm (9.4–13.4 in) circumference at upper arm (Compare to small adult or standard adult size blood pressure cuff)

Forearm strap: 13.5"; Upper arm strap: 18"

CATALOG ITEM:

**REF** 959XL-Ultra



(01)00686004000736


**LOT** 2024-04-01-959XL-Ultra REV 0424




# I.V. House 959XL-Ultra TLC® Elbow Splint Directions

Single Use Only—Fits either arm.


The TLC Elbow Splint can be applied to the arm before or after inserting IV and connecting to extension set or IV tubing.

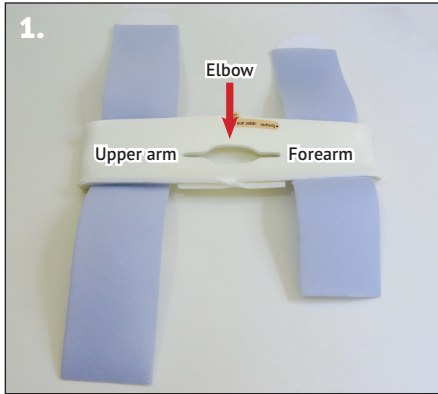

1. Remove splint from package and unwrap straps. With foam side up, note orientation indicated on label. Then peel label away and discard.

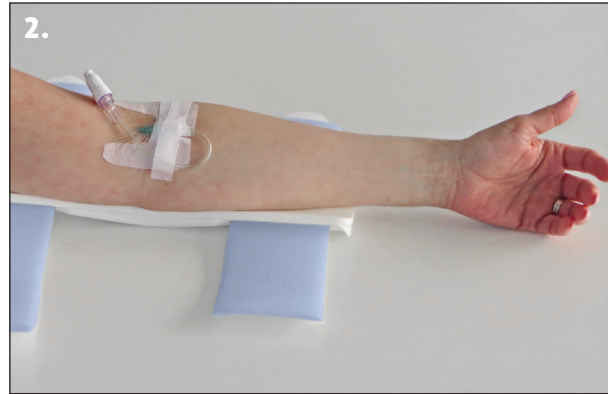


2. With patient's thumb pointing up, fit foam surface of splint to patient's arm. Align point of patient's elbow to center opening in splint. Place wide end of splint at patient's upper arm, narrow end at forearm.

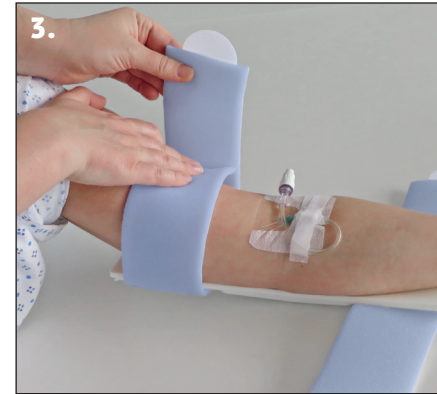


3. Wrap strap at wide end of splint around patient's upper arm.



4. Secure strap with Velcro® coin.

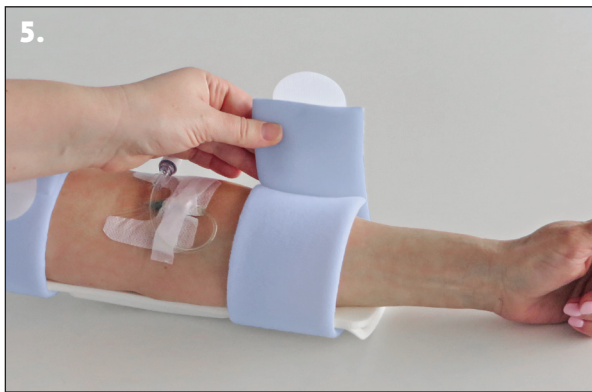

5. Wrap strap at narrow end of splint around patient's forearm.

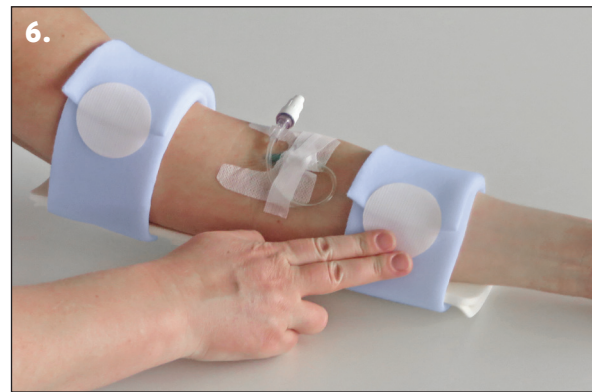

6. Secure strap with Velcro® coin.



7. Protect I.V. insertion site with a 730Arm I.V. House UltraDressing®.